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CONTENT / DESCRIPTION

CryoPump® ankle brace :
 - 1 positioning and retaining strap,
 - 1 ON/OFF valve,
 - 2 gel packs,
 - 1 air pump (+ tube) with velcro fixing,
 - 1 unique, personalised protective pouch.

Product information leaflet, to be read carefully.

INDICATIONS

Treatment of post-operative and post-traumatic damage to ligaments, Rehabilitation.

COMPOSITION

Knee brace :
 - Nylon 32.5 %
 - Polyurethane 25 %
 - Polyethylene 35 %
 - Silicone 5 %
 - PVC 2.5 %

Cold pack :
 - Nylon 50 %
 - Cellulose 50 %

PRECAUTIONS

This device must be fitted and adjusted by a health professional (pharmacist, orthopaedic specialist, physiotherapist, etc.). If you experience any pain or swelling or notice any change in the way it feels or any unusual symptoms, seek medical advice immediately. Do not wear the device next to the skin. Cover any part of the body where an analgesic may have been applied before applying the device. It is strongly advised that the device is not used for any other patient.

CLEANING

- Handwash in soapy water (max. 30°C),
 - Rinse thoroughly,
 - Allow to dry naturally,
 - Do not tumble dry,
 - Do not bleach,
 - Do not use detergent.

Store at room temperature.

FITTING INSTRUCTIONS**FIRST USE :**

Place the two gel packs inside the protective pouch and leave it in the freezer for 2 hours before using.

1 Fig. 1 - Fig. 2

- Ensure the air pump is secured firmly to the valve.
 - Place the gel pack in the desired position.

2

- Make sure the brace is facing the right way, with the logo visible.

3 Fig. 3 - Fig. 4

- Slide foot into the splint ensuring that your heel is correctly inserted into the recess.
 - Close the ankle brace by wrapping the strap around the ankle.

4 Fig. 5

- Check the valve is in the ON position and pump until you reach the desired pressure.
 - If necessary, reduce compression by pressing on the pump button.
 - Close the valve (OFF position).

5

- Depending on medical advice, repeat the process with the second gel pack and return the used pack to the freezer.

NOTE :

The pump can be attached to the ankle brace using the Velcro fixing or removed by unscrewing it from the valve.

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